

# Getting Started Guide



## Tobii Eyegaze

### Overview

- The Tobii Eyegaze is a peripheral eye tracker that replaces the standard mouse, allowing you to navigate and control a desktop or laptop computer using only your eyes.
- This device can open up a world of fun, communication and control for those users that are unable to use anything other than their eyes to operate the PC.

### Step by Step

To connect it to PC

1. The Eyegaze unit plugs into the PC by USB cable and sits on the front of the screen using a magnetic connector.

To set up a user:

2. Open the Eyegaze Interaction software by clicking on the red eye icon:



3. This will launch the gaze settings screen:



5. Click on "Track Status" to check if the Eyegaze is positioned so it can track your eyes. You may need to move up, down or nearer or further
6. Once you are positioned correctly, select "Calibrate" and ask the user to follow the moving circle with their eyes.
7. Once Calibration has completed, you can either click OK at the bottom of the screen to start using the Eyegaze or click on "User Profile" from the left side navigation menu to create a profile and save that calibration to that user.

### Where to find more information

- Watch the videos and tutorials here: <http://www.tobii.com/en/assistive-technology/global/education/training/?product=17397>
- Full user manual : <http://www.tobii.com>