


# Getting Started Guide



## Vibe-Lite Switch

### Overview

- Round illuminated switch, designed for tactile and visual stimulation.
- It can be used alone as a sensory tool or used with any program or other game or toy that is designed to use a switch such as the "switch it" games suite.
- Select from momentary, latch or timed and connect your equipment.
- Adjustable time from five seconds to two minutes.
- Blink feature illuminates once every four seconds, illuminates and vibrates when pressed (both features can be switched off if required).

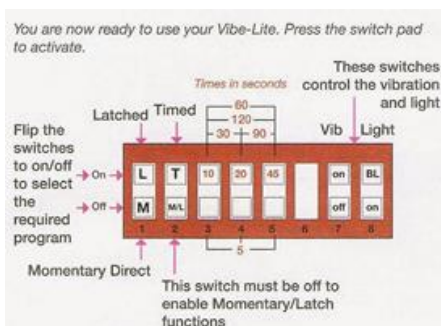

### Step by Step

1. The switch is connected to the universal switch box



2. This switch box has 4 input options. Connect the vibe-light to whichever socket matches your required function.



3. The switch box connects to the PC via a USB socket. Once connected ensure the MODE light is switched OFF by depressing the button
4. To activate the switch press down firmly on it.
5. To change the switch settings on individual games, remember to go into the "settings" or "options" menus of the games.
6. If you are experiencing problems with the switch working, check the connections and the batteries.

### Where to find more information

<http://senict.blogspot.co.uk/2014/05/review-of-vibe-lite-switch-from-switch.html>