

# Getting Started Guide



## myGaze 2 – using myGaze EyeMouse Play

### Overview

myGaze® Assistive 2, created by Visual Interaction, is gaze control of computer programs made easy and affordable.


- Easy Start
- No-fail calibration
- Works like a mouse with most software
- Suitable for early as well as power users


### Step by Step

To connect it to PC

1. The Eyegaze unit plugs into the PC by USB cable and sits on the front of the screen using a magnetic connector.


To open eyegaze software

1. The myGaze EyeMouse Play software should automatically open in calibration screen display when turned on but if not then double click the icon to open. Next go to settings, and then under the calibration tab click calibrate. 
2. The positioning window will appear to allow correct positioning. A coloured surround will indicate correct positioning.

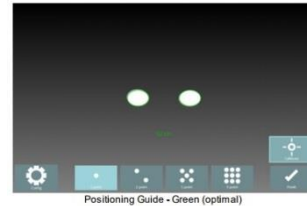

There are three status displayed; green - correct position, orange - better position, red – incorrect position. Arrows will appear to direct the user to be in the centre of the screen

3. From this window you can then decide on a 1, 2, 5 or 9 point calibration. Click Calibrate to start the calibration.
4. The myGaze Menu provides access to the myGaze EyeMouse Play functions

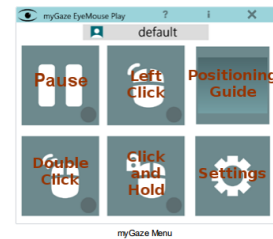

5. Go to settings to configure settings for specific users and conditions.
6. **TIP** The software is running if there is a rectangle at the top of your desktop screen. The colour of this rectangle indicates correct positioning.



### Where to find more information

- Watch the video here: <https://youtu.be/4uJZWolpCDY>
- Full user manual : [http://www.mygaze.com/fileadmin/download/mygaze\\_assistive2/myGaze\\_EyeMouse\\_Play\\_User\\_Guide.pdf](http://www.mygaze.com/fileadmin/download/mygaze_assistive2/myGaze_EyeMouse_Play_User_Guide.pdf)