

Getting Started Guide




iSwitch for iPad

Overview

A new bluetooth switch which enables switch access to your iPad

Step by Step

1. Ensure Bluetooth is turned on from your iPad. (Settings, Bluetooth)
2. Ensure the iSwitch is charged. Charging point (D)
3. Turn on the iSwitch using the Channel button (F)
4. The iSwitch should appear as a discoverable device on your iPad. Tap on the name and the pairing process will begin. It takes about 20seconds to connect after which your iPad will state that the device is connected.

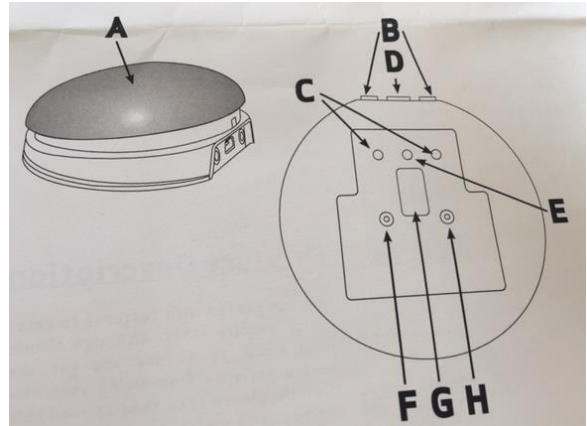

5. The default modes for the integral switch (A) is SPACE, Socket 1 (B) is ENTER and Socket 2 (B) is Home. Socket 1 and 2 will accept most wired switches.

6. **To change the function of the integral switch (A)** - Press the Channel button (F) once. The current mode setting will be displayed (the table indicating the function). To change, press the mode button (H) until the desired setting appears on the display.

(Note neither of the LEDs are illuminated (C) when the integral switch mode is altered)

7. **To change the function of either socket 1 or 2** – first select the socket you would like to change by repeatedly pressing the channel button (F) until the LED (C) adjacent to the relevant socket is illuminated.

The current mode setting will be displayed. Press the mode button (H) until the desired setting appears on the display