

# Getting Started Guide



## Using Wired Switches with Inclusive Technology Multiswitch Box

### Overview

Switches are a great way to allow a user with limited motor skills to play games, have fun, learn, take control and express themselves.

Lifelites supply a number of different switches and software packages that work well with switches.

The difficulty can sometimes be getting the settings right on the switches for the program you are using them with but the Multiswitch box is very simple to use

### Step by Step

#### Start Up

1. Connect your Multiswitch Box to the PC using the USB cable.

2. Ensure the  button is NOT highlighted

3. The 6 numbered ports have the following functions:



- 1 – Space
- 2 – Enter
- 3 – (this is not programmed)
- 4 – (this is not programmed)
- 5 – Left click
- 6 – Right click

(these are marked on the front of the box)

4. Connect a wired switch into the relevant port for the game you wish to play

Unfortunately there is no universal standard for switch functions and some programs will use space and enter, some left mouse click and right mouse click and others other functions.

If you are not sure what the settings should be, try the game or activity using the mouse or keyboard and see what works. You can then put the switch in the correct port accordingly.