

Getting Started Guide




BIG Step-by-Step

Overview

BIG Step-by-Step Choice with Levels provides an exciting option for easy-tech, quick-ready communication. The device has 4 minutes of recording time.


Step by Step

1. Turn the device on by rotating the volume On/Off knob

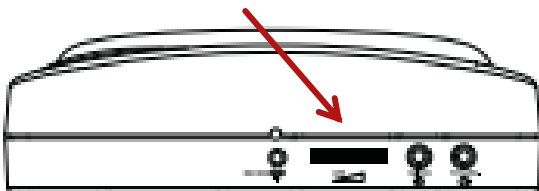


2. The device has 3 levels to choose from. Select the desired level with the switch on the side.

Level choice

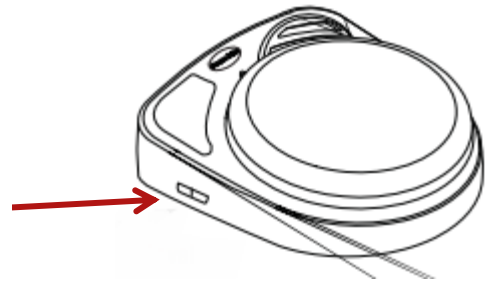

3. Press and hold the record button until the Record light activates, then release.

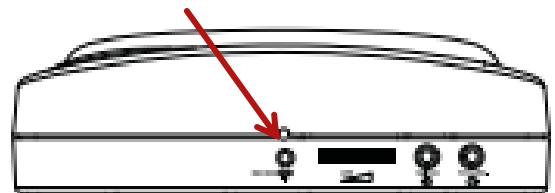

4. Press and hold the switch top, begin speaking when you hear a beep and release when finished. Repeat for additional messages in sequence.



5. Press and release the record button to exit record mode.
6. To play messages – Press the switch top to play a message. You can advance through the messages by pressing the switch top again. Select the level to hear the other messages recorded.

BATTERIES REQUIRED This device takes a 9V square battery