


Getting Started Guide



Vibe-Lite Switch

Overview

- Round illuminated switch, designed for tactile and visual stimulation.
- It can be used alone as a sensory tool or used with any program or other game or toy that is designed to use a switch such as the "switch it" games suite.
- Select from momentary, latch or timed and connect your equipment.
- Adjustable time from five seconds to two minutes.
- Blink feature illuminates once every four seconds, illuminates and vibrates when pressed (both features can be switched off if required).

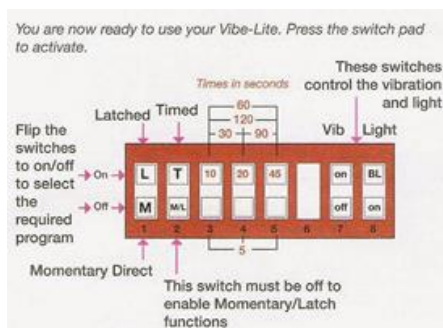

Step by Step

1. Connect your Multiswitch Box to the PC using the USB cable.
2. Ensure the mouse button is NOT highlighted
3. The 6 numbered ports have the following functions:



- 1 – Space
- 2 – Enter
- 3 – (this is not programmed)
- 4 – (this is not programmed)
- 5 – Left click
- 6 – Right click

(these are marked on the front of the box)

4. To activate the switch press down firmly on it.
5. To change the switch settings on individual games, remember to go into the "settings" or "options" menus of the games.
6. If you are experiencing problems with the switch working, check the connections and the batteries.

Where to find more information

<http://senict.blogspot.co.uk/2014/05/review-of-vibe-lite-switch-from-switch.html>