

Getting Started Guide



Candy Corn switch

Overview

- The Candy Corn Proximity Sensor Switch is highly sensitive and requires zero pressure to activate.
- Just place your hand or other part of your body within 10 millimeters of the switch to activate.
- Visual and auditory feedback when activated and an internal magnet for mounting.

Step by Step

1. The switch is connected to the Beamz universal switch box.
2. Connect your Beamz universal switch box to the PC using the USB cable.
3. Ensure the mode button is NOT highlighted



4. The 4 numbered ports have the following function:

- 1 – Space
- 2 – Enter
- 3 – Left click
- 4 – Right click

(these are marked on the reverse of the box)

5. Connect the candy corn switch into the relevant port for the game you wish to play.
6. Turn the switch on using the on/off button on the reverse.
7. To activate the switch just position your hand or other part of your body within 10 millimeters of the switch to activate.



8. To change the switch settings on individual games, remember to go into the “settings” or “options” menus of the games.

9. If you are experiencing problems with the switch working, check the connections, and check the battery (it takes a CR2032 battery).

